

SALTER LABS®

Salter Labs® *Percent-O₂-Lock® Venturi System*

Single unit variable venturi system delivering accurate concentrations of oxygen (24%, 26%, 28%, 31%, 35%, 40%, 50%) with a single unit.



Features

- ☐ Transparent, air-entrainment mask with soft anatomical form
- ☐ Easily adjusted, variable venturi system
- ☐ Unique, springloaded mechanism locks prescribed concentration in place
- ☐ Wide, adjustable elastic strap or over-the-ear style available in adult configuration
- ☐ Durable, plastic humidity cup, standard

Benefits


- ☐ Ideal for long-term users and edentulous patients. Better fit.
- ☐ Accurate concentrations of oxygen with single unit
- ☐ Ensures safety and correct oxygen concentrations
- ☐ Fits comfortably in either configuration to hold mask in place
- ☐ Allows utilization of a room air humidifier or nebulizer with mask when prescribed




slc-53

Percent-O₂-Lock® Venturi System P/N 8150

The Percent-O₂-Lock® air entrainment oxygen mask includes a rotating sleeve, opening and closing an air entrainment window. This provides the ability to deliver seven (7) fixed oxygen concentrations to a spontaneously breathing patient. Concentration levels are easily dialed and locked in. The dial-in air entrainment assembly is affixed to a standard aerosol mask. A high humidity adaptor permits use of a nebulizer with mask if prescribed.

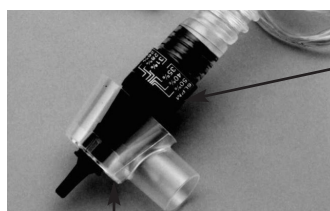


All units are pre-assembled with the dial pre-set at 24%

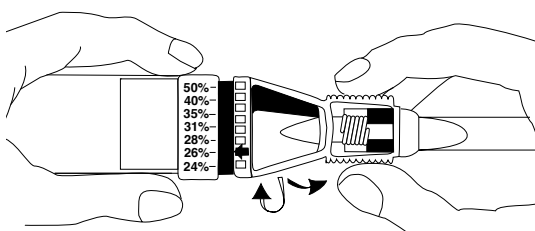

To change a setting, pull the sleeve back until the spring is completely compressed. Then turn the sleeve to the prescribed setting.

The high humidity adaptor fits over and is pushed on until it locks on the post extending away from the tip of the cone.

Caution: THE HUMIDIFYING OR NEBULIZING DEVICE MUST BE DRIVEN BY COMPRESSED AIR.

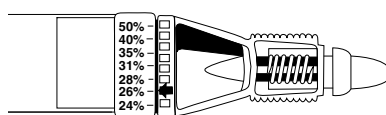

Releasing the sleeve and spring forces the sleeve into its setting. The spring also locks the sleeve into place

Ordering Information



Percent-O₂-Lock® Venturi System*

	Units (per case)	Elastic-Strap Style	Over-The-Ear Style
Adult Percent-O ₂ Lock® air entrainment system with humidity cup, 7' oxygen supply tube, and mask	50	8150	8050
Adult Percent-O ₂ -Lock® /trach tee air entrainment system with humidity cup and 7' oxygen supply tube	50	8054	
Adult Percent-O ₂ -Lock® air entrainment assembly with humidity cup (without mask or oxygen supply tube)	50	8056	
Adult Percent-O ₂ -Lock® air entrainment assembly (without mask, oxygen supply tube or humidity cup)	50	8057	
Adult Percent-O ₂ -Lock® air entrainment assembly with 7' oxygen supply tube and humidity cup (without mask)	50	8058	
Adult Percent-O ₂ -Lock® air entrainment assembly with 7' oxygen supply tube (without mask or humidity cup)	50	8059	
Pediatric Percent-O ₂ -Lock® air entrainment system with humidity cup, 7' oxygen supply tube, and pediatric mask	50	8121	

SALTER LABS®

100 W. Sycamore Road - Arvin, CA 93203
 Phone: 661-854-3166
 1-800-421-0024
 1-800-235-4203
 Fax: 661-854-3850
 Toll Free Fax: 1-800-628-4690
 Fax: 661-854-3850
 E-mail: salterlabs@us.salterlabs.com
 www.salterlabs.com